



Laparoscopy

Specimen retrieval bag with handle / without handle

WWW.BODY-PRODUCTS.DE

medi-inn® PRO Specimen retrieval bag with handle

Infinitely variable intraoperative control with integrated handle

MEDI-BAG GRIP supports the infinitely variable intraoperative opening and closing of the specimen retrieval bag in the abdomen. This is controlled using the pusher handle, which pushes the self-opening specimen retrieval bag through the opening in the insertion sheath. The specimen retrieval bag is closed securely by pulling the pusher handle and thread.

MEDI-BAG GRIP – With handle

- Self-opening and releasable specimen retrieval bag with integrated handle
- Transparent, highly tear-resistant, and liquid-tight retrieval bag made of thermoplastic polyurethane (TPU) for the protected collection of small and large resected tissue
- Kink-resistant Nitinol wire with memory effect for secure intra-abdominal closure by pulling the integrated thread
- Colour-coded Medi-Bag handles for quick orientation when choosing the suitable bag volume, ranging from 200 to 1500 ml
- Extremely easy to use thanks to infinitely variable intraoperative opening and closing
- 10 mm trocars for simple insertion of the retrieval bag
- Latex-free
- Single-use
- Double-packed in sterile packaging

Applications

- Adrenalectomy
- Appendectomy
- Cholecystectomy
- Cystectomy
- Ectopic pregnancy
- Laparoscopic colon surgery (bowel, sigma, and colon resection)
- Lymph node removal
- Nephrectomy
- Prostatectomy
- Thoracoscopy (extraction of lung segments)



Retrieval bag volume	Retrieval bag diameter	Trocar diameter	Packaging unit	Product code
200 ml	80 mm	10 mm	5	BP-40350006
500 ml	100 mm	10 mm	5	BP-40350007
800 ml	130 mm	10 mm	5	BP-40350008
1200 ml	130 mm	10 mm	5	BP-40350009
1500 ml	130 mm	10 mm	5	BP-40350010

medi-inn® PRO Specimen retrieval bag without handle



Easy operation and gentle application


The **MEDI-BAG** model without a handle is particularly easy to use. The Medi-Bag specimen retrieval bag is pushed into the abdomen through the insertion sleeve using the integrated pusher and expands in a matter of seconds after being released from the insertion sheath. The specimen retrieval bag is closed using grasping forceps.

MEDI-BAG – Without handle

- Self-opening specimen retrieval bag with insertion sheath and pusher
- Transparent, highly tear-resistant, and liquid-tight retrieval bag made of thermoplastic polyurethane (TPU) for the protected collection of small and large resected tissue
- Coated, kink-resistant Nitinol wire with memory effect for secure intra-abdominal closure
- Bag volumes from 200 to 800 ml
- 10 and 12 mm trocars for simple insertion of the retrieval bag
- Latex-free
- Single-use
- Double-packed in sterile packaging

Applications

- Adrenalectomy
- Appendectomy
- Cholecystectomy
- Cystectomy
- Ectopic pregnancy
- Laparoscopic colon surgery (bowel, sigma, and colon resection)
- Lymph node removal
- Nephrectomy
- Prostatectomy
- Thoracoscopy (extraction of lung segments)

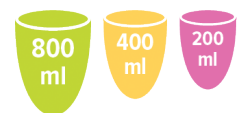

Retrieval bag volume	Retrieval bag diameter	Trocar diameter	Packaging unit	Product code
200 ml	50 mm	10 mm	10	BP-40350011
400 ml	90 mm	10 mm	10	BP-40350012
800 ml	100 mm	10 mm	10	BP-40350013



Your contact:

BODY PRODUCTS
Relax Pharma und Kosmetik GmbH

Alfred-Nobel-Straße 1-3
50226 Frechen

Phone: +49 2234 91165-0
Fax: +49 2234 91165-23
Email: info@body-products.de

WWW.BODY-PRODUCTS.DE